

Indications for Use

QuikBow® is a pin tensioner for skeletal traction.

The QuikBow TM is comprised of

- 1. a pin tensioner for 2mm (5/64th inch) pins
- 2. a sterile rope to hang traction (10ft)

Warnings, QuikBow®

- 1. The surgeon should have specific training, experience, and familiarity with the use of K-wires for skeletal traction.
- 2. The device is designed to work optimally with 2mm or 5/64th inch smooth K-wire and if a different diameter pin is used, the device may not grip the pin.
- 3. Tighten the tension screw and confirm pin tension before weight is hung from the bow.
- 4. QuikBow® is single use only, do not reprocess.
- 5. For prolonged traction, periodically confirm pin tension. More tension can be applied by tightening the tension screw.

Instructions for Use, QuikBow®

- 1. Place a 2mm pin in the distal femur or proximal tibia, adhering to accepted standard of care principles.
- 2. Confirm the pin is placed in the desired location.
- 3. Turn the blue tension screw to set the desired span, up to 9.5"
- 4. Compress blue thumb levers and apply QuikBow® to the pin.
- 5. Tighten the tension screw as hard as possible to apply desired tension to the pin.
- 6. Confirm pin has been tensioned before applying weight.
- 7. Loop traction rope around the tension screw handle to hang desired traction weight, up to 40 lbs.
- 8. Consider elevating the direction of pull to keep the QuikBow® off of the patient's tibia, or pad the tibia.

Sterile Product

QuikBow® is supplied sterile, Prior to use, inspect the package for damage, which may compromise sterility. If damaged, the product must be assumed to be non-sterile.

DO NOT USE IF STERILE PACKAGE IS DAMAGED. DO NOT USE AFTER EXPIRATION DATE.

MRI Safety Information

The QuikBow® has a designation of MR Unsafe and is not acceptable for MRI.

Federal Law (USA) restricts this device to sale by or on the order of a physician.

Symbols Glossary

Symbols	Description
REF	Reference Number
LOT	Batch Code
	Do not use if package is damaged
	Do not reuse
R _x only	Device only to be sold on or by the order of a physician.
	Manufacturer
	Caution
	Consult Instructions for Use
	MR Unsafe