

Nailbed Repair SwiftKit™

Instructions for Use



Indications for Use

Nailbed Repair SwiftKit™ is indicated for nail bed repair procedures.

The Nailbed Repair SwiftKit™ is comprised of

- 1. QuikLift™ Sterile Freer Elevator, and
- 2. General accessory items

Warnings

- 1. Single use only. Do not reuse.
- 2. The surgeon should have specific training, experience, and familiarity with the procedure before attempting to use the Nailbed Repair SwiftKit™.
- 3. Contraindications: Patients not indicated for nail bed repair procedures.
- 4. Nailbed Repair SwiftKit™ is supplied sterile and should be unpackaged using sterile technique. The product is sterile if packaging is undamaged and unopened. Prior to use, inspect the package for any damage which may compromise sterility. **DO NOT USE AFTER EXPIRATION DATE.**

Instructions for Use, Nailbed Repair SwiftKit™ for nailbed repairs.

- 1. Gather the items not included in the kit: gloves, antiseptic, anesthetic, and suture.
- 2. Examine the injury and plan approach.
- 3. Unpack the Nailbed Repair SwiftKit™ using sterile technique and establish the sterile field.
- 4. Soak and irrigate the wound, clean the hand and perform block as indicated.
- 5. Apply tourniquet to the finger and irrigate the wound again if needed.
- 6. Use QuikLift™ Freer to elevate the nail.
- 7. Perform repair using suture or dermabond.
- 8. Stent-open the eponychial fold to prevent it from scarring down using nail or synthetic material. Secure stent with suture.
- 9. Remove tourniquet and dress the finger with Xeroform and gauze. Secure with tape.

Symbols Glossary

Symbols	Description
	Reference Number
	Batch Code
	Do not use if package is damaged
	Do not reuse
	Device only to be sold on or by the order of a physician.
	Manufacturer
	Caution
	Consult Instructions for Use
	MR Unsafe