

Foot Debridement SwiftKit™

Instructions for Use



Indications for Use

Foot Debridement SwiftKit™ is indicated for distal extremity debridement and repair.

The Foot Debridement SwiftKit™ is comprised of

- 1. QuikRev™ Sterile Bone Rongeur
- 2. QuikLift™ Sterile Freer Elevator, and
- 3. General accessory items

Warnings

- 1. Single use only. Do not reuse.
- 2. The clinician should have specific training, experience, and familiarity with the procedure before attempting to use the Foot Debridement SwiftKit™.
- 3. Contraindications: Patients not indicated for extremity debridement and repair.
- 4. Foot Debridement SwiftKit™ is supplied sterile and should be unpackaged using sterile technique. The product is sterile if packaging is undamaged and unopened. Prior to use, inspect the package for any damage which may compromise sterility. **DO NOT USE AFTER EXPIRATION DATE.**

Instructions for Use, Foot Debridement SwiftKit™ for extremity debridement.

- 1. Gather the items not included in the kit: gloves, antiseptic, anesthetic (if applicable)
- 2. Examine the injury and plan approach.
- 3. Clean and disinfect the wound and surrounding skin.
- 4. Unpack the Foot Debridement SwiftKit™ using sterile technique and establish the sterile field.
- 5. Probe and assess the wound.
- 6. Debride wound as needed using included instruments.
- 7. Irrigate and clean the wound and apply dressing.

Symbols Glossary

Symbols Description	
	Reference Number
	Batch Code
	Do not use if package is damaged
	Do not reuse
	Device only to be sold on or by the order of a physician.
	Manufacturer
	Caution
	Consult Instructions for Use
	MR Unsafe